PRIMEDIC™

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Modular defibrillator equipment.
 Made by PRIMEDIC™.
 Conceived for medical emergencies.
 Considering our product-philosophy „keep it simple and safe“.

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GB

The convincing advantages:

- ❑ PRIMEDIC™ -defibrillators are quickly ready for operation.
- ❑ Self-test check constantly
 - high-voltage unit
 - charging time
 - exactitude of the energy charge
 - state of the accumulator.
- ❑ Pictograms and a self-explanatory program flow allow a secure handling.
- ❑ automatic functions prevent maloperations.
- ❑ achieves a diagnosis within seconds and a faultless operation.
- ❑ all connector plugs are protected in the housing.
- ❑ slot for additional modules integrated in the housing. (only DM 10 and DM 30)
- ❑ weight of the defibrillator including paddle and accumulator less than 8.5 kg.
- ❑ a highly shock-resistant plastic housing protects the defibrillator during emergency operation.
- ❑ excellent product for the medical security.
- ❑ used 10,000 times world-wide.

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GB

**The Defi family and its
operational area:**

Hospitals

- resuscitation room · emergency admission · angiogram room · intensive care department · ergometric department · operating theatre · rehabilitation · diff.departments/wards

Consultants

- cardiologist · internal specialists · general practitioners · surgeons · anesthesiologists

Rescue services

- Red Cross · Maltese and St. John's Ambulance · diff. private rescue services · fire-brigade-armed forces

Lay rescuers

- fire-brigade · police · sports clubs... everyone

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The superior security
for medical practices,
clinics

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GB

Emergencies do not
tolerate any compromises.

**PRIMEDIC™ Defi-N
– for mains operation is
a defibrillator of the
latest generation and
stands for a fully-
developed and reliable
technology.**

Its handling is safe and
easy.
PRIMEDIC™ Defi-N is
immediately ready for
operation. After
switching it on, the
device performs an
internal self-test with
error-detection
automatically.
Pictograms and a self-
explanatory program
flow ensure a secure
handling. The clearly
structured operating and
display elements inform
the user unmistakably
about the operating
state of the defibrillator.

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GB

**PRIMEDIC™ Defi-B
– accumulator operated
defibrillator**

The mobile security which proves its worth daily - being mains-independent - in ambulance cars, in cardio-sportgroups, in clinics etc.

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Every second counts in case of a cardiac arrest. The PRIMEDIC Defi-B is in a position to reassure the rescue party by allowing an extremely fast defibrillation readiness, an easy handling and a well tested technology.

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Extremely rugged and reliable.

PRIMEDIC™ Defi-B is ready for operation quickly , even if sockets are out of reach, thanks to the high accumulator capacity.

After having switched it on, the defibrillator performs an internal self-test considering all important parameters. Pictograms and a clear program flow ensure a functional handling.

The paddle cables are kept safely in the housing.

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The well-structured user guidance may be life-saving.

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The PRIMEDIC Defi-B can easily be transported and is ready for operation everywhere immediately.

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Paddles and cables are kept neatly in the defibrillator

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GB

Extremely economical accumulator operated defibrillator with monitor.

The convincing advantages:

- clearly structured operating elements
- flat housing, protected connectors and paddle cables
- memory of the last 10 defibrillations
- high-resolution printer
- highly contrasting and big LCD-monitor, tiltable at a range of 30°
- synchronous/asynchronous operating mode
- 3-channel-ECG, 1-channel-ECG-printer
- long accu-operating time (about 2.5 h monitoring or 60 shocks)
- 5 seconds charging time for 360 joule

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Original-printout
1-channel
25 mm printing
speed

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Tilttable monitor: constantly high visibility - no reflection

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GB

PRIMEDIC™ Defi-monitor DM 1/ DM 3

Offering all the advantages of the ECO 1, still with the latest EL-monitor generation; highest contrast and an excellent readability at the same time.

The convincing advantages:

- big, highly contrasting EL-monitor
- excellent readability, it may be looked at from all sides (170° angle of view)
- 6-channel-EKG, 1/3-channel-EKG-printer

Defi-monitor DM 3 with NELLCOR®-pulse-oximetry

- The DM 3 is equipped with the SpO₂-monitoring of the world market leader NELLCOR®
- all current NELLCOR® SpO₂-sensors may be used.
- a pictorial representation of the pulse curve on the monitor and in case of 1-channel-printing

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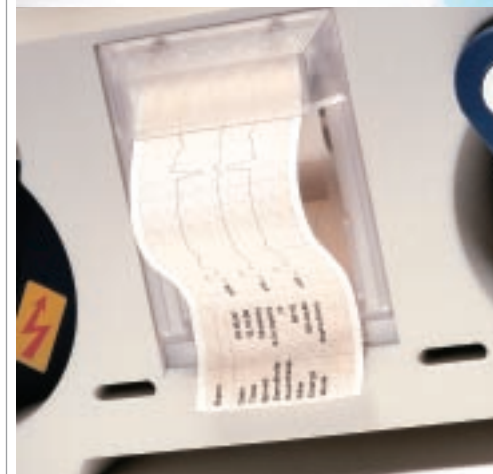
Original-printout
3-channel, 50 mm
printing speed

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The accumulator may be detached and exchanged at any time by pushing a button.

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GB

Defi-monitor DM 10 / DM 30

- Accumulator operated manual defibrillation.
- Synchronous/asynchronous operating mode for internal defibrillation
- With the big and highly contrasting EL-monitor and with all the advantages of the Defi family, as e.g. a simple and secure user guidance and a very flat housing
- Front-to-back size only 12 cm
- 6-channel-ECG with erasure bar.

The convincing advantages:

- additional to model DM 1/3, a modular slot for a lateral connection of different PRIMEDIC™ -modules as e.g. PRIMEDIC™ pacer-module, I/D module and R/D module
- extended operating panel
- 6-channel-ECG and 1/3-channel-printer

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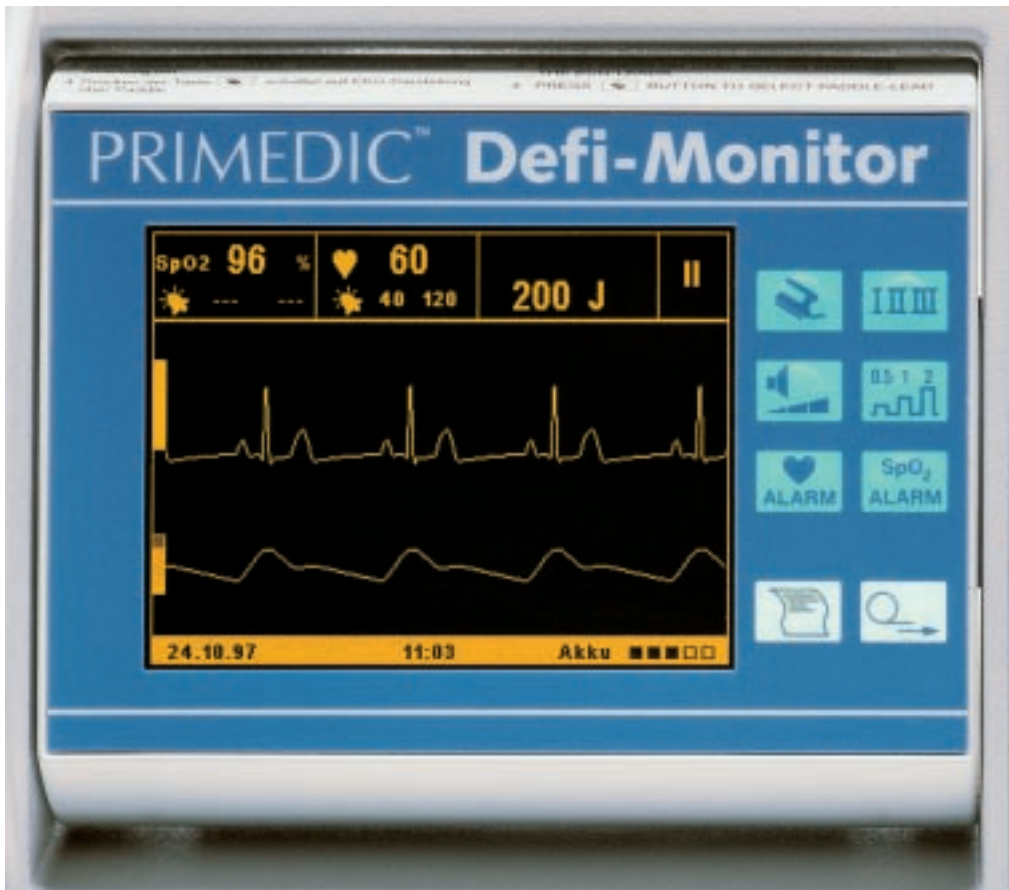
The easily-surveyed membrane keyboard with well-arranged self-explanatory operating elements.

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The clear and highly contrasting EL-monitor guarantees an excellent readability from any angle of view.

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GB

The Defi-Monitor DM 30 including NELLCOR®-Pulse oximetry

- The pulse curve is presented on the monitor as well as on the printout
- wide range of NELLCOR®-sensors for pulse oximetry
- three SPO₂ alarm functions are adjustable during the setup
- the degree of oxygen saturation is shown on the display
- the SPO₂-sensor determines the heart rate (as long as the ECG-cable has not been plugged in.)

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GB

An independent jury of renowned professors and doctors chose our Defi-monitor among 52 products to be the superior one.

„Honoured as an exemplary product for the medical security during the contest „Goldene SMM '97“

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☐ with all operating modes:

- FIX
- DEMAND
- OVERDRIVE

☐ Pulse range of the pacer from 30 up to 250 pulses per minute

☐ Surprisingly simple and flexible handling

The convincing advantages

☐ Defibrillation may either be made via paddles or via adhesive electrodes

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GB**PRIMEDIC™-I/D-module**

Module for connecting internal spoons. For internal defibrillation.

Energy: 2/5/7/10/20/30/50 joule, automatic restriction of energy to a maximum of 50 joule, energy adjustable at the defibrillator.

The convincing advantages:

- the additional equipment for internal defibrillation may be used as a module at any time without having to carry out constructional modifications
- unnecessary purchasing can be prevented at a time when the exact operative range is not yet known.

PRIMEDIC™-R/D-module

Module for the connection of adhesive electrodes for remote defibrillation, incl. adhesive electrodes and defibrillation cable

The convincing advantages:

- allows freehand defibrillation without having to use the paddles
- ideal contact of the adhesive electrodes and riskless application of the shock energy at the same time

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Pacer and cable are stored neatly and handily in the defibrillator

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GB

Active as well as passive safety elements are important components of all our systems.

The charging console of the Defi-monitor ensures in an extremely space-saving way the permanent preparedness in ambulance cars, medical practices or clinics. The charging console cares for a constantly loaded accu, keeps the optional stand-by accu ready for operation at any time and prolongs its service-life considerably by the optional Accu-Care-Unit. Therefore, you cannot only rely on a 100 % charge, but also on a 100 % capacity.

The convincing advantages:

- Charging function for Defi- and stand-by accu
- Changeable accu with high capacity
- Charging time of only 45 minutes for a fully charged accu
- automatic and safe locking of the defibrillator without having to use additional fixation elements
- perfect installation in vehicles (no huge metal constructions are required)

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PRIMEDIC™ -charging
console, for charging
and maintaining a
2nd accu

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PRIMEDIC™-Accu-Care
table top device,
mains-operated device
for loading and main-
taining an extra accu

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GB

- we have no problem with a lack of room: space-saving and hygienical wall fastening in medical practices and clinics
- the accu gets optimally maintained by means of „Accu-Care“
- Double-contacts at the charging console are gold-coated and do only become alive when the defibrillator has been set up

Extreme power needs care

Nickel-cadmium-accus show convincing advantages: they are quickly charged, offer about 3000 charging cycles and ensure a high capacity at a low weight. Typical of NiCd-accus: the memory effect. If they have only partly been discharged, the available capacity of the accu decreases considerably.

This may only be prevented by a complete discharge of the accu after each operation. Still, in daily use this may hardly be put into practice, as to the accu of the defibrillator it is nearly impossible. PRIMEDIC™ Accu-Care prevents the memory effect by selective charge- and discharge current.

PRIMEDIC™ Accu-Care is available as slide-in module for the charging console and as a separate table-top device.

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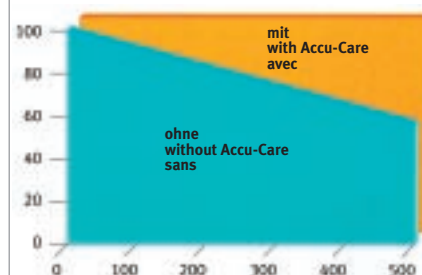
The Defi-monitor may be unlocked from the charging station with only one grasp

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For the internal defibrillation we dispose of a large range of different internal spoons

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GB

A wide range of excellently harmonizing accessories is an essential component of a practice-orientated product design. We dispose of different attachment modules for the charging console, special sensor- and ECG extension cables as well as the important SpO₂-sensor-family of the world market leader NELLCOR®.

It goes without saying that we pay special attention to our accessories, always considering our product philosophy „Keep it simple and safe“. The original-PRIMEDIC™-accessories ensure a maximum of security at a fair price.

□ ECG-disposable electrodes Ø 55 mm, Ag/AgCl, with push button are available in packaging sizes of 30 pcs/unit.

□ Adhesive electrodes as combined electrodes for pacing and defibrillation for children and adults in pairs

□ Complete kit for internal defibrillation incl. 2 patient cables for internal defibrillation
 2 handles for internal spoons
 2 internal spoons for adults

□ SpO₂-finger sensor DS-100 A, NELLCOR® for multiple use, for adults (>40 kg)

□ SpO₂-Y-sensor D-YS, NELLCOR® for multiple use with ear clip. Further SpO₂-NELLCOR®-sensors are available

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Two of the most frequently used NELLCOR-sensors for the pulse oximetry: finger sensor and universal-y-sensor with ear clip

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GB**Practical transportation- and accessories bag with compartments**

Practical accessories bag with 3 compartments offers room for everything you need. Plug-in of the ECG cable in rest condition, for example, saves important seconds during emergency situations.

- For all Defi-monitors
- Transportation bag with 2 compartments for Defi-N and Defi-B incl. strap.
- For all Defi-monitors
- Strap with fixation elements and shoulder pad

Further accessories + cables

- ECG-patient cable, 4-pole with integrated Defi-protection, neutral electrode
- Extension cable (5 m) for ECG-patient cable, for vehicle installation
- SpO₂-patient cable (2,4 m) for connection with NELLCOR®-sensors, incl. traction relief
- Extension cable (5 m) for SpO₂-patient cable, for vehicle installation
- Pacer-defibrillation-cable for connecting adhesive electrodes
- 12/24 V-connection cable (2 m) suitable for an installation in the vehicle
- Special cables and special lengths are available on demand
- Anchoring clip, robust traction relief for ECG- and SpO₂-patient cables, for attaching to clothes
- Patient cable for internal defibrillation, sterilisable with screw thread
- Handles for internal spoons

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Cushion electrodes: Extensive and flexible cushion electrodes with a very low transient resistance for pacing and freehand defibrillation.

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GB

Since our foundation in 1973, we have con-centrated ourselves on the strength of German development technique: Continuous innovation, highest quality demands and a high investment in research and develop-ment (about 10 % Metrax-internal rate). This has resulted in precise and highly reliable high-tech products, combined with an outstanding user-orientation.

This combination helped us to establish our products worldwide, always considering our product philosophy: Keep it simple and safe.

In clear:: PRIMEDIC™-defibrillators have an intelligent design and are well-proved in medical practice. More than 10,000 defibrill-ators sold worldwide as well as several awards show the high accep-tance of our products in the international business.

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trademark of Metrax GmbH,
NELLCOR® is the registered trademark
Metrax GmbH